



FreeStyle

Freedom Lite

Blood Glucose Monitoring System

1. Wash hands.
2. Hold lancing device.
3. Insert lancet.
4. Turn dial.
5. Turn dial 3x.
6. Press lancing device.
7. Remove lancet.
8. Prick finger (1-4 drops).
9. Insert strip.
10. Add blood.
11. Insert strip into meter.
12. Meter displays result.
13. Remove strip.
14. Dispose of strip at $> 45^\circ$.
15. Meter beeps.
16. Meter displays 5.8.
17. Dispose of lancing device in sharps container.

Abbott Diabetes Care

Klantenservice

0800 - 022 88 28 (gratis)

 **Abbott**
Diabetes Care